


* Partial drilling on half the length of the implant.

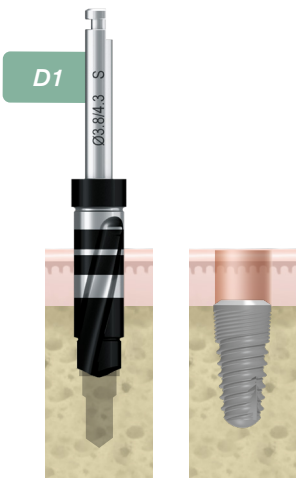

Use the cortical drill whatever is the bone density.

..... Example of insertion for a 10 mm long implant, the same as for the other lengths of implants.

IMPLANTS Ø 4.5

Lengths 6 - 8 - 10 - 12 - 14 mm


D4	D3 -D2	D1
	Ø 3.8	Ø 3.8

STAGED DRILL

STAGED DRILL

CORTICAL DRILL

VERY HARD BONE DRILL

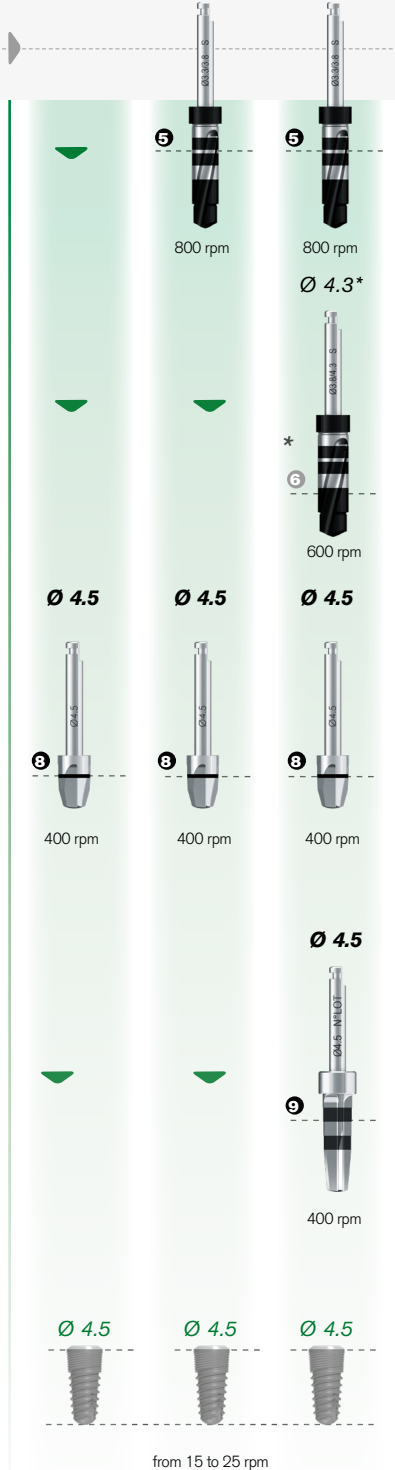

IMPLANTS Ø 5

Lengths 6 - 8 - 10 - 12 - 14 mm

D4	D3 -D2	D1
	Ø 4.3	Ø 4.3

